

Preformed and rounded tip

Ideal space solution

With only 3 clicks from 20 to 65 cm



Pocket Introducer

The smallest packaged introducer to facilitate difficult intubation

/ Pocket Introducer

Pocket Introducer is an aid for the difficult airway to facilitate orotracheal intubation under direct laryngoscopy or via video laryngoscopy. The 15 Fr introducer with preformed tip and centimetre markings unfolds from 20 to 65 cm with only 3 clicks and ready for use within seconds.

In folded condition, the introducer provides the smallest pack size and is the ideal space solution for emergency bags, helicopters, lab coat pockets, crash carts, etc.


Use



In folded condition, the Pocket Introducer measures 20 cm.



Unfold the Pocket Introducer.

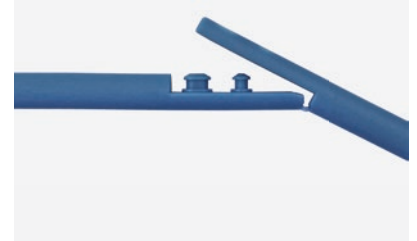

Firmly press together all click connections.




The click connections are permanent and cannot be detached again.



If necessary, shape the distal end.



Preformed and rounded olive tip is atraumatic.

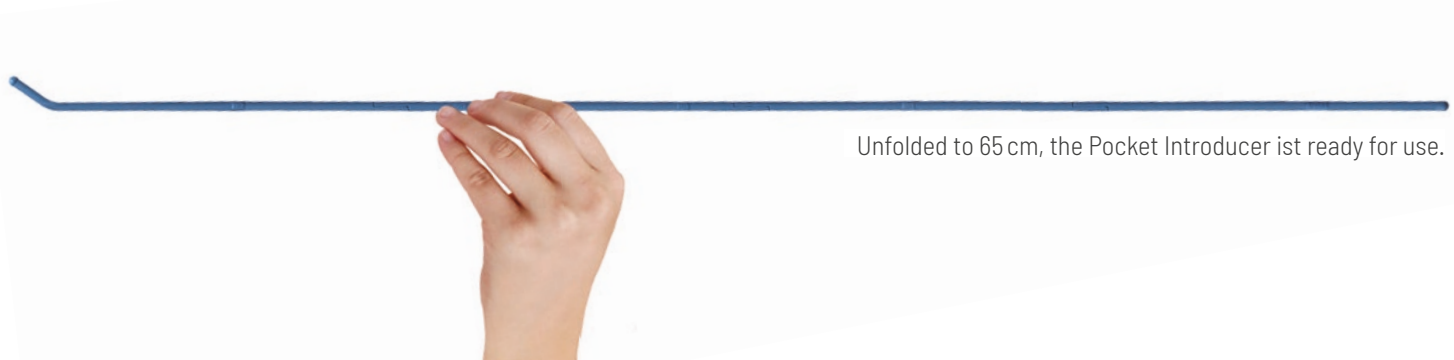

Unfolded to 65 cm, the Pocket Introducer is ready for use.

Order information

Pocket Introducer / For single use, sterile

Size	Length	Centimetre Markings	For Tracheal Tube	REF	Box
15Fr	65cm	10 to 40 cm	≥ 6.0 mm I.D.	33-80-650-1	5

Pocket Introducer / In robust plastic cartridge, for single use, sterile

Size	Length	Centimetre Markings	For Tracheal Tube	REF	Box
15Fr	65cm	10 to 40 cm	≥ 6.0 mm I.D.	33-81-650-1	1

Additional information



Application video



Flyer intubation aids



Annotated bibliography



Plastic cartridge

This product is manufactured without the use of natural rubber latex, unless otherwise specified.
This product does not contain phthalates which require labelling according to CLP Regulation (EC) 1272/2008.